


Settings in AudBase for GSI Audiometer and Tympanometer

1. From Subject List Screen
 - a. Select File from top Toolbar
 - b. Select “Preferences”
2. From Preferences Window
 - a. Select “Audiology (3)”
 - b. Click “Assignments” Button on right side of window
 - c. The following Window will appear

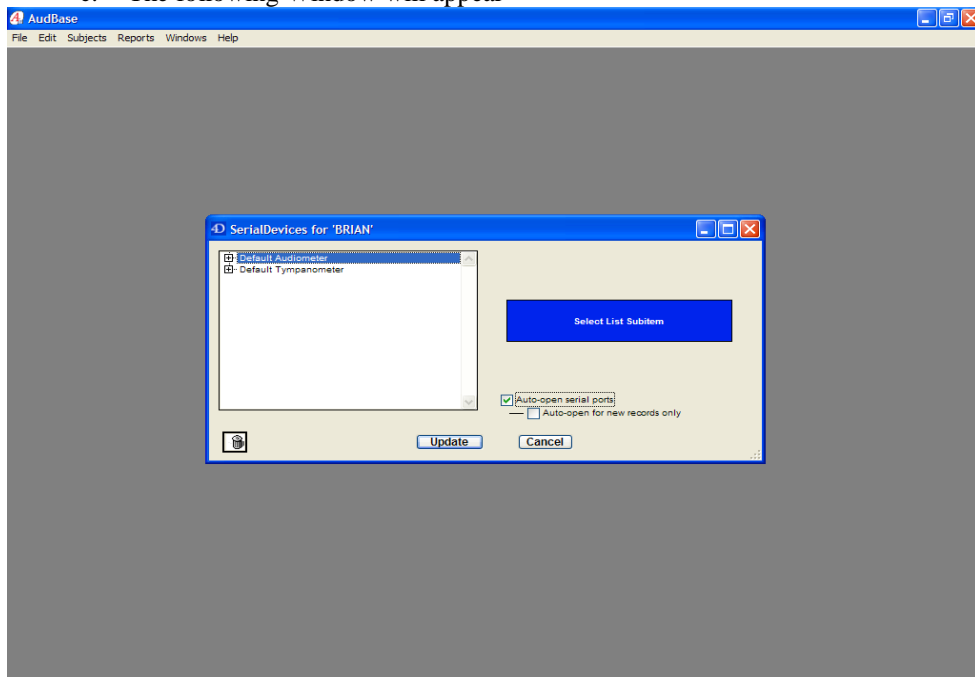

3. Click the “+” to the left of “Default Audiometer”
 - a. There will be three settings
 - i. Device Name
 1. In right window, select GSI 16/61
 - ii. Serial Port
 1. In right blue window drop down, select com port the Audiometer is attached to
 - a. Only active com ports on computer will be available
 - iii. Communication Protocol
 1. Protocol: DTR, Buad Rate: 9600, Data Bits: 8, Stop Bits: 1, Parity: None
 - b. Make sure Check mark is in box “Auto-open serial ports”
 - c. Click “Update Settings” in Blue Box
4. Click the “+” to the left of “Default Tympanometer”
 - a. There will be three settings
 - i. Device Name
 1. Select either Tymptstar or GSI33 from right blue window drop down menu
 - ii. Serial Port
 1. In right blue window drop down, select com port the Tympanometer is attached to
 - a. Only active com ports on computer will be available
 - iii. Communication Protocol
 1. Protocol: DTR, Buad Rate: 9600, Data Bits: 7, Stop Bits: 2, Parity: Even
 - b. Make sure Check mark is in box “Auto-open serial ports”
 - c. Click “Update Settings” in Blue Box
5. Select “Update” Button at bottom of window
6. Close Preferences Window

- When transferring data, make sure to first hit the “REMOTE” button on the keypad of the unit, “Remote” will appear on the display
- On the GSI-33, after you press the “REMOTE” button, select soft key for “Record Type” and choose “Summary + Graphics”
- On Tymptstar, press “ETF” button, then “RETURN” button.
 - Select “INSTRUMENT OPTIONS” softkey
 - Select “DATA TRANSFER SETTINGS” softkey
 - For “Tymp” change setting to “SUMMARY + GRAPHICS”
 - Select “STORE”